
ProTaq[®] Decalcifier

quartett's portfolio includes different decalcifier which have been found useful for various purposes. Choosing a decalcification solution depends on three main factors:


1. Type of sections to be cut.

2. Type of staining to be performed.


3. Speed of process required.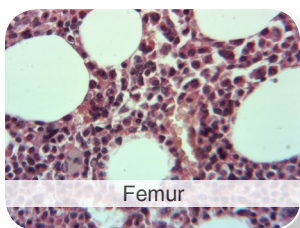

Femur

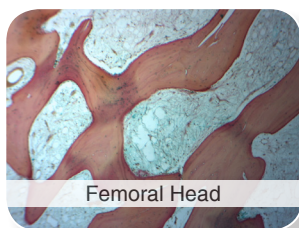
Femoral Head

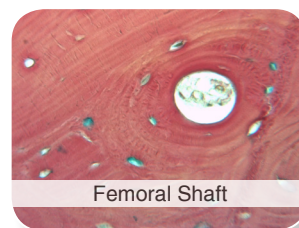
Femoral Shaft

Tibial Artery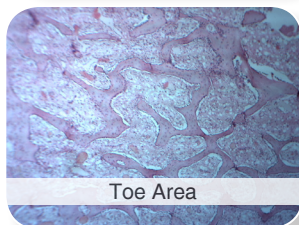

Toe Area

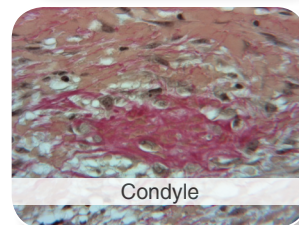Condyle

ProTaq[®] CALLESS PLUS

Longterm decalcification

Due to its mild formulation the solution is suitable for all sample types ranging from small needle biopsies to dense cortical bone. Depending on the sample size the decalcification process can be carried out overnight. Important details such as cell structures for diagnostics are not destroyed or influenced. All types of routine stains work perfectly.

Catalog number: 401605701

Quantity: 1 Liter

ProTaq[®] CALLESS FAST

Fast decalcification

This product is a quick decalcifier, which is often used for bone marrow samples. Normal bone marrow biopsy is decalcified within one hour. The product can be used for all types of routine stains. Cellular as well as nuclear structures are clearly preserved.

Catalog number: 401605801

Quantity: 1 Liter

ProTaq[®] CALLESS HISTO

Mild decalcification for IHC

This mild descaling solution was developed specifically for immunohistochemistry. Bones are decalcified slowly and gently without destroying the antigens. The solution is suitable for bones of all kinds.

Catalog number: 401605901

Quantity: 1 Liter

ProTaq[®] CALLESS FIX

Fixation and decalcification

Good decalcification depends on a good fixation. This solution does both in one run. The product was specifically developed for large dense bones, which are difficult to fix. As soon as the solution begins to descale the samples, it allows better penetration of formaldehyde. Cortical bones, femoral heads and teeth are better decalcified due to better fixation.

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